

Hello!

Must tell you about what is happening at the anticoagulation center in Norrköping. There, there is a great staff group who since some time now is using Zafena's Simple Simon PT and Journalias AVK-letter, photos of the work area are shown below.

The Norrköping staff is very happy with the new routines. Patient safety has increased as there are now no manual transfers of the INR-results into the AVK-letter. Also, the direct access to the AVK-letter at the point of care increases the quality of the discussions about the treatment with the patient. The routine works like this:

*The patient comes into the room, blood is drawn from a finger and the assay is started.**The patient file is retrieved and displayed on the screen by use of the barcode on PT-control-letter that the patient is carrying.***While the INR-assay is in progress, it takes about one minute, the patient can observe the progression on a smaller screen while discussing medications, diet alterations and possible bleedings, information that is registered "behind the buttons" in Journalias AVK-letter. ****As the INR-analysis is completed, the results are retrieved into the AVK-letter by use of the "Dos-button".*****The dose is then decided, the "letter" printed, and the patient leaves the room with the PT-control-letter in hand.

With these new routines the overall time requirement has decreased considerably, for patient as well as for staff. This is mostly because everything is done in one go while the patient remains in the room.

If you want to know more, contact Zafena, the organization that develops and sells Simple Simon PT, phone +46 141 405 20, info@zafena.se , or, of course, contact us at Journalia, phone +46 554 22231.

Yours sincerely, Anna Ramebäck, Journalia AB, anna.rameback@journalia.se



