


Approved by:	Version: 20230828
Date:	Author: FW <span style="float: right;">Page 1 by 3</span>

## Eurolyser CUBE and CUBE-S Instructions

The document describes the connection and labeling of the meter, to work with the Zafena POC-Workstation. Physical connection is between USB-A and USB-B.

Product code cable: **ZAF-520-3**

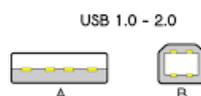

### Instrument-tag: LMC-ECS1




Print and attach the instrument-tag on to the meter. In some cases, to enhance the work flow, it might work better to attach the tag in close proximity to the meter.

### Communication settings

1. Choose "Settings"  
*Menu pictures (from CUBE VET) for communication settings are the same for all CUBE models.*




2. Choose "Interface"

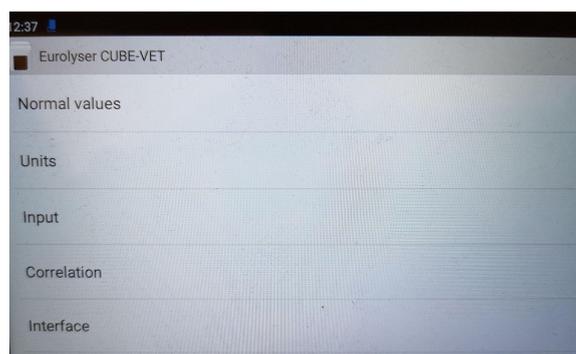


Approved by:	Version: 20230828
Date:	Author: FW <span style="float: right;">Page 2 by 3</span>


3. Choose “Host”

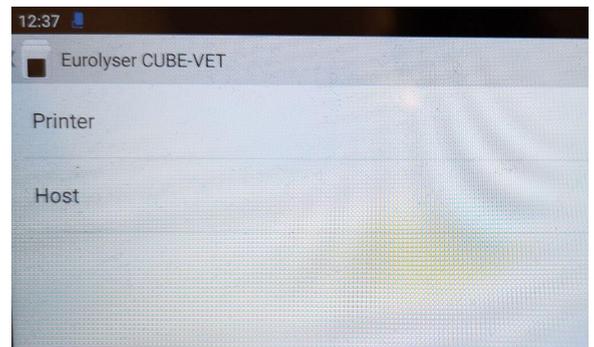

4. Type in password:  
**3030**

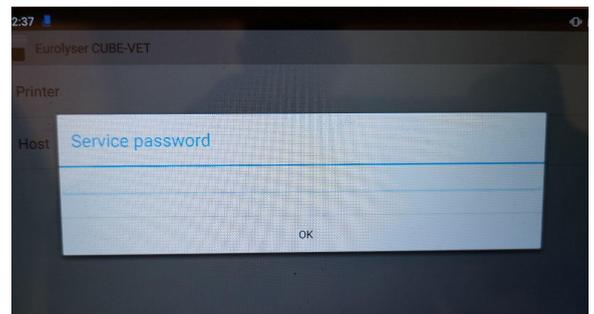


5. Choose “EMR/LIS”

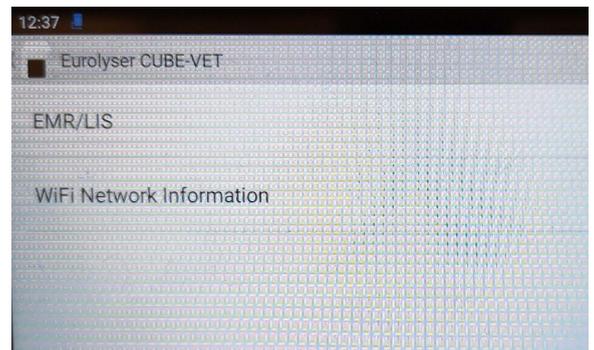

6. Set these parameters to:  
EMR Protocol “TEXT/PLAIN USB”  
Automatic transmission: “Yes”  
Transmit operator ID: “Yes”  
Transmit lot number: “Yes”  
Transmit serial number: “Yes”

Then back out of this menu and go back to the start menu.

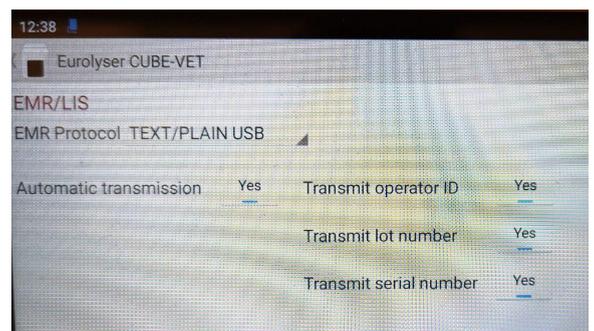



**Eurolyser CUBE  
Instrument-tag**

Approved by:	Version: 20230828
Date:	Author: FW <span style="float: right;">Page 3 by 3</span>

It is now ready to transmit.