

|              |   |
|--------------|---|
| Approved by: | Version: 20230907   |
| Date:        | Author: FW <span style="float: right;">Page 1 by 1</span> |

## Installing a new Micro-SD card into the Zafena POC-Workstation [[ZAF-552](#)]

### In short


The document describes how to replace or install a new memory card into the Zafena POC-Workstation. You should have the new memory card (Micro-SD card) at hand. Be observant not to mix the memory cards.

### Preparations

1. Note the current software version here or on any piece of paper. You locate the version number in the lower left corner of the Zafena POC-Workstation display. You will use the version number, at the end, to verify the update.



2. Turn off the Zafena POC-Workstation by pressing and letting go of the ON/OFF button on the right side of the casing. The shutdown is complete, when the light on the button stops flashing.

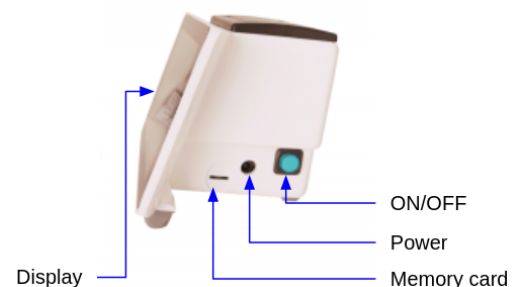

3. Remove the old memory card by gently grabbing the card from below, with a thin paper clip or something alike. There is a tiny bar on the bottom of the card surface that will enable you to pull it out. Put the old memory card aside.
4. Install the new memory card in the same slot as the old, with its goldish contacts surfaces facing up. Insert it at a slightly upwards slanting position. The card should hook into a slide and fully submerge into the slot.
5. Start the Zafena POC-Workstation by pressing and letting go of the ON/OFF button. It will start to flash and after a while the light will stay solid.
6. Check the software version number and compare it to the version number you wrote down.