

MICROSEMI CRP celleteller

Dokumentet beskriver hvordan Microsemi CRP kobles til Zafena Work Station (ZAF 552) med spesial kabel for serial oppkobling

Instrument-tag



Barkoden festes på instrumentet

Sett i spesial kabel


Artikkel nr ZAF-523
pl2303 USB - RS232 med noll-modem F



Innstilling instrumentet

16.2.3 RS-232C settings

- **Setting method**
- **Communication conditions**
Data rate: 9600 bps; Data length: 8 bits; Stop bit: 1; Parity: None
One-way communication only, from the instrument to the PC or other device.
Flow control and handshake are not performed.
- **Configuring instrument settings**
(Refer to "11.1.4 RS-232C data output settings" (page 141).
Perform the following operation to turn ON the RS setting:
[Menu] -> [The Other Functions] -> [Settings] -> [Function 1] -> [RS] -> [ON].

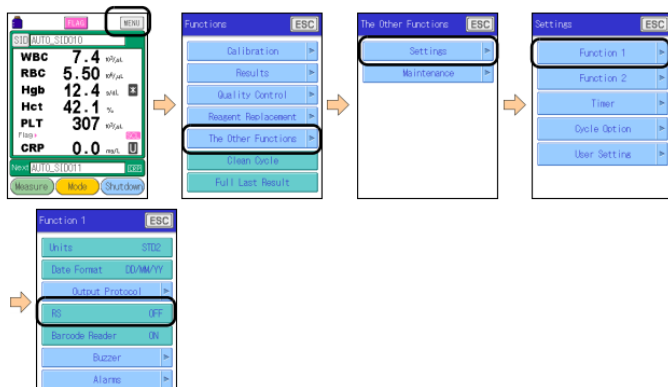

1. Trykk menu, høyre hjørnet
2. The other functions
3. Settings
4. Functions 1
5. RS = ON

