

|                 |  |
|-----------------|--|
| Approved by: XR | Version: 20241108  |
| Date: 20241108  | Author: NKR <span style="float: right;">Page 1 by 2</span> |

## Skyla VB1 Instructions

The document describes the connection and labeling of the meter, to work with the Zafena POC-Workstation. Physical connection is between USB-A and RS232.

Product code cable: **ZAF-523**



### Instrument-tag: LMC-SVB1

Print and attach the instrument-tag on to the meter. In some cases, to enhance the work flow, it might work better to attach the tag in close proximity to the meter.



### Communication settings:

1. Choose "Settings" and then "Administrator Settings"  
(Universal password 5787722)



2. Choose "RS232 Baud Rate"




- Set Baud Rate to: **"115200"**  
Press "OK" to confirm



- Scroll down in the settings-menu using the gray arrow  
Choose "Automatically Send"




- Set Automatically Send to: "ON"  
Press "OK" to confirm

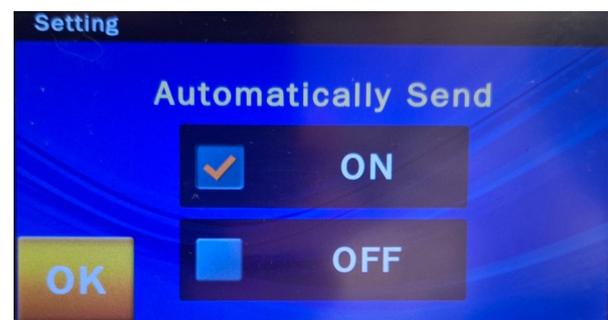

- Choose "Protocol"  
Set Protocol to: "skyla"  
Press "OK" to confirm



- Choose "Interface"  
Set Interface to: "RS-232"  
Press "OK" to confirm

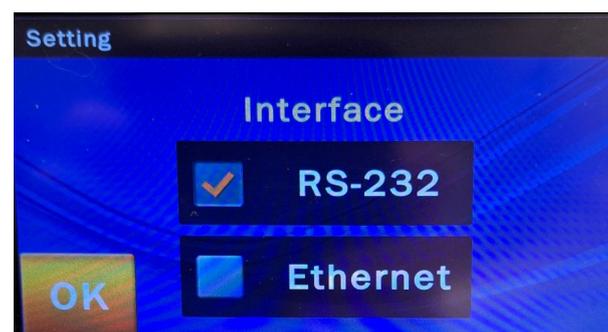

- Press the house icon to get back to the home screen.
- Follow the quick guide to start your digitalization.